


SOLEM™ Vein Holder

Providing a gentle, firm grasp
of vein grafts for precise positioning
and suturing.

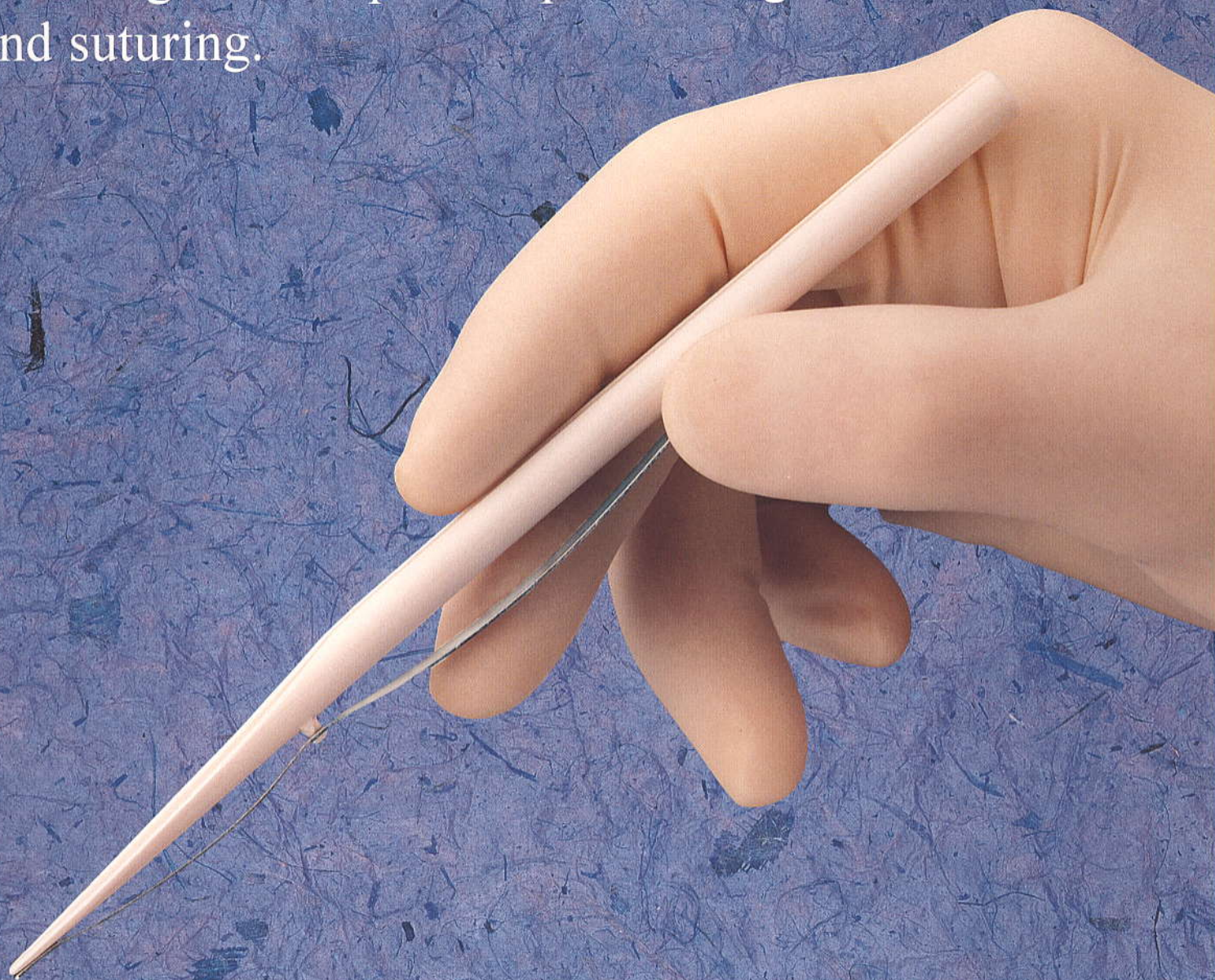


SCANLAN ★

INTERNATIONAL

Surgical products designed and manufactured by the Scanlan family since 1921

SOLEM™ Vein Holder

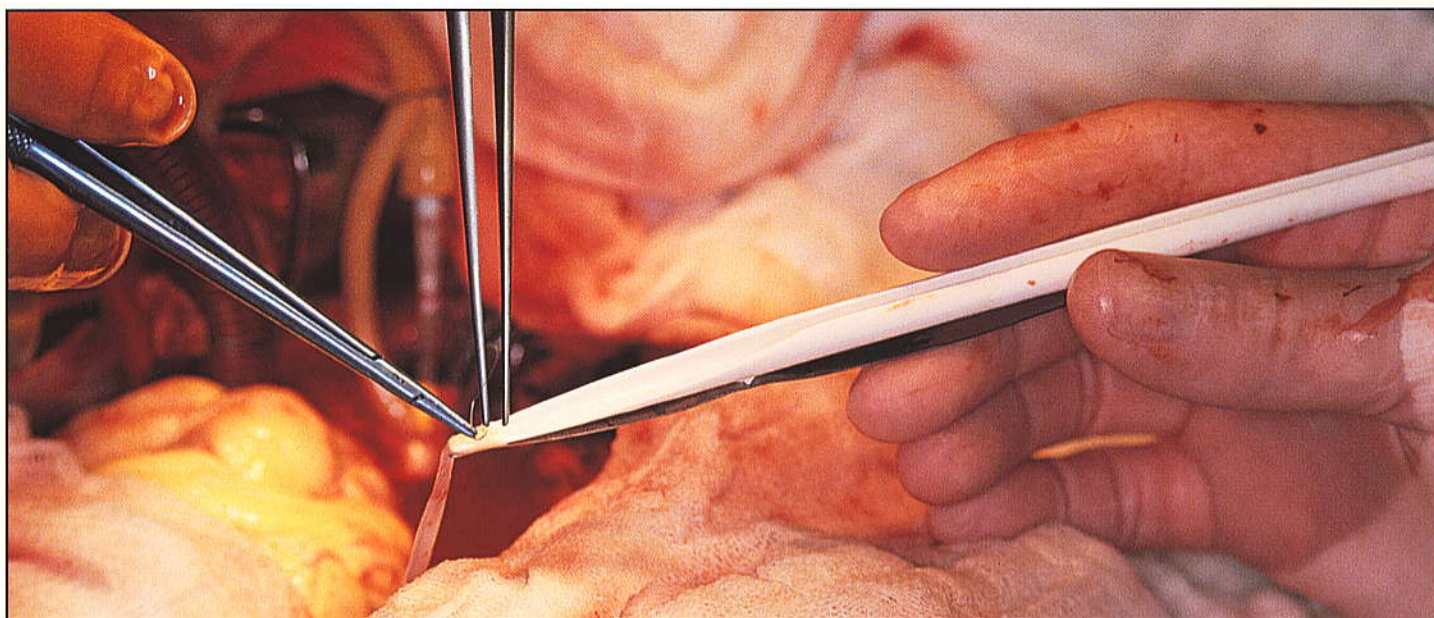

PAT. 34,854,318

The Scanlan® SOLEM™ vein holder offers a unique design to aid the surgeon during vein graft placement. Unlike conventional methods, it provides a gentle, but firm grasp of vein grafts and allows for precise positioning and suturing. The slim profile and concave tip of the Scanlan® SOLEM™ vein holder reduces the trauma that can occur with other vein graft placement methods. In addition, the Scanlan® SOLEM™ vein holder...

- Features a smooth streamlined, concave tip to assure optimal exposure.
- Minimizes the need for assistance while setting sutures.
- Has a lightly serrated clip that provides steady, consistent pressure to minimize graft movement.
- Prevents catching of the opposite vein graft wall during suturing.
- Reduces trauma by working without the use of pins, hooks, or teeth.
- May be used with various size grafts due to its tapered tip.
- Lightweight.
- Single-Use.
- Radiopaque.
- Latex-Safe.

INSTRUCTIONS FOR USE:

The vein holder is opened by depressing the stainless steel clip to accept the vein graft. The instrument is gently inserted into the lumen of the vein graft with the metal clip providing light pressure on the underside of the vein. Once positioned on the vein holder the vein graft is ready to accept sutures. Rotating the vein holder allows suture placement around the graft's circumference. As the sutures are drawn into place, the vein graft is gently removed from the vein holder.

CATALOG NUMBER

1001-760

LENGTH

6 1/2" / 16.5 cm

For more information or to place an order,
please contact your local Scanlan representative, or call...

U.S.A. & Canada: 800-328-9458

International: 651-298-0997

FAX: 651-298-0018 • www.scanlangroup.com

SCANLAN ★

INTERNATIONAL

Surgical products designed and manufactured by the Scanlan family since 1921

One Scanlan Plaza • Saint Paul, Minnesota 55107 • USA
800-328-9458 • 651-298-0997 • FAX 651-298-0018 • www.scanlangroup.com

© 2001 Scanlan international, Inc.
MK-B-029 Rev. A